

# LTV<sup>®</sup> External Power Accessories



## LTV Transport Battery System

**Redefining portable ventilation.** At roughly one-third the weight of comparable sealed lead acid batteries, the Lithium Ion battery was designed to maximize the portability of the LTV Series ventilators. While weighing only 2.7 lbs (1.2 kg), each battery provides an average of three hours of portable external power. Further enhancing mobility, the LTV Transport Battery System integrates both the ventilator and battery into one rugged, soft side Transport Pack, making it ideal for hospital, air and ground transport.

Part Number: 17332-001 or 17563-001

## Universal Power Supply

**Continuous uninterrupted power.** The Universal Power Supply provides added assurance against power outages and interruptions. Designed to connect directly in-line between the AC power source and the LTV Series ventilator, the unit continuously recharges its four-hour backup battery, as well as the ventilator's internal battery. In the event of loss of AC power, the ventilator will automatically switch over to the Universal Power Supply's backup battery without interruption to patient ventilation, making it ideal for use in hospital, rehab, long-term care and home care settings.

Part Number: 14546-001



## 3-Hour External Battery Kit (12V/12Ah)

**Convenient LTV portability.** Weighing less than 9 lbs (4 kg), the 12V/12Ah sealed lead acid battery provides an average of three hours of portable external power to the LTV Series ventilator. The battery kit features a padded nylon bag with shoulder strap for optimal portability, as well as a battery charger capable of recharging a fully depleted 12V/12Ah battery within four hours.

Part Number: 10900

## 9-Hour External Battery Kit (12V/33Ah)

**External power with rugged durability.** Our longest lasting battery, the 12V/33Ah sealed lead acid battery provides an average of nine hours of additional external power. The kit features a durable, pre-wired battery box, which allows the battery to be mounted directly to a wheelchair or the LTV floor stand for added convenience. A fully depleted 12V/33Ah battery can be recharged within eight hours.

Part Number: 10800



# LTV<sup>®</sup> External Power Accessories



	Lithium Ion Battery	Universal Power Supply	3-Hour External Battery	9-Hour External Battery
<b>PHYSICAL SPECIFICATIONS</b>				
Weight:	2.7 lbs (1.2 kg)	21.6 lbs (9.8 kg)	8.6 lbs (3.9 kg)	26.5 lbs (12 kg)
Height:	1 in (2.5 cm)	8.5 in (21.6 cm)	4.1 in (10.4 cm)	7.1 in (18 cm)
Width:	6 in (15.2 cm)	9.2 in (23.4 cm)	4 in (10.2 cm)	5.2 in (13.2 cm)
Depth:	9.3 in (23.6 cm)	5.6 in (14.2 cm)	6 in (15.2 cm)	7.7 in (19.6 cm)
<b>POWER SPECIFICATIONS</b>				
Battery Chemistry:	Lithium Ion	Sealed Lead Acid	Sealed Lead Acid	Sealed Lead Acid
Operating Time:	3 Hours	4 Hours	3 Hours	9 Hours
Charge Status Indicator:	5 Segment LED	5 Segment LED	N/A	N/A
Nominal Capacity:	9 Ah	20 Ah	12 Ah	33 Ah
Nominal Voltage:	14.4V	12V	12V	12V
Recharge Time:	4 Hours	4 - 6 Hours	4 Hours	8 Hours
<b>TIPS FOR OPTIMAL BATTERY LIFE</b>				
	During long-term storage, store the battery at half capacity. Avoid storing for more than 6 months, as this can decrease the useful life of the battery.	Connect to an AC power source whenever possible. Frequent charging will prolong the life of the unit's internal battery.	Always store the battery with a full charge. Storing in an uncharged state can permanently decrease the battery's capacity.	Whenever the battery is not in use, keep it connected to the charger. Frequent charging will help prolong the life of the battery.
<b>RELATED ACCESSORIES</b>				
	LTV Transport Battery System Dual Battery Option (P/N 17332-001)	Universal Power Supply (P/N 14546-001)	3-Hour Battery Kit (P/N 10900)	9-Hour Battery Kit (P/N 10800)
	LTV Transport Battery System Single Battery Option (P/N 17563-001)	UPS Floor Stand Mounting Bracket (P/N 15009-001)	3-Hour Battery & Bag (P/N 10910)	9-Hour Battery & Box (P/N 10866)
	Lithium Ion Battery (P/N 14497-001)	UPS Replacement Battery (P/N 17894-001)	3-Hour Battery (P/N 11475)	9-Hour Battery (P/N 10787)
	Lithium Ion Battery Charger (P/N 15084-001)		3-Hour Battery Cable (P/N 10901)	Battery Box (P/N 10790)
	LTV Transport Pack (P/N 15096-001)		Padded Battery Bag with Cable (P/N 11330)	Battery Charger (P/N 10801)
			Battery Charger (P/N 10801)	Battery DC Cord (P/N 10802)
			Battery DC Cord (P/N 10802)	LTV Floor Stand Battery Mounting Bracket (P/N 11454)